## PATIENT INSTRUCTION GUIDE

**1. INTRODUCTION** The guidelines about how to handle, clean and maintain your contact lenses must be primarily informed by your Eye Care Professional. The correct asepsis of your contact lenses is important since it avoids complications improving the comfort and visual acuity. For your eye health, it is important that your contact lenses be worn only as prescribed by your Eye Care Professional. These lenses must be monthly replaced or as recommended by your Eye Care Professional. These lenses are valid for one (1) month after the packaging is opened. Your Eye Care Professional should be kept fully aware of your medical history and will develop a total program of care based on your specific needs. **2. WEARING RESTRICTIONS AND INDICATIONS** Soft, Soloxiflex Contact Lenses are intended to enhance and/or alter the appearance of natural eye and also for correction of myopia as prescribed by the Eye Care Professional. The lenses are intended for daily wear or flexible wear (use during the day and remove to sleep) for the period of one month or as recommended by the Eye Care Professional. They are not intended for use when asleep. **3. CONTRAINDICATIONS** There are certain health conditions with which contact lenses should not be worn. You should alert your eye care professional to any of the conditions listed below. Only your eye care professional can determine if contact lens wear is right for you.

1. Inflammation or infection in or around the eye or eyelids. 2. Any eye disease, injury or abnormality that affects the cornea, conjunctiva, or eyelids. 3. Any previously diagnosed condition that makes contact lens wear uncomfortable. 4. Severe dry eye. 5. Reduced corneal sensitivity. 6. Any chronic disease that may affect the eye or may be made worse by wearing contact lenses. 7. Allergic reactions on the surface of the eye or surrounding tissues that may be induced or made worse by wearing contact lenses. 8. Irritation of the eye caused by allergic reactions to ingredients in contact lens solutions (i.e., rewetting drops). These solutions may contain chemicals or preservatives (such as mercury, Thimerosal, etc.) to which some people may develop an allergic response. 9. Any active eye infection. 10. If eyes become red or irritated. 4. **PRECAUTIONS** Do not use multipurpose solution during the period they are not in use. 8. The users must be instructed by the eye care professional about the procedures to place and remove the contact lenses. 9. Do not use saliva to hydrate the contact lenses. 10. Keep the fingernails cut short. Do not use sharp objects while handling the contact lenses. 11. Avoid using contact lenses in environments subject to irritating vapors. 12. Never apply eye medications while using your contact lenses. 13. If you experience any discomfort, heavy tearing, pain or red eyes, sensitivity (light sensitivity), immediately remove the lenses from your eyes and consult your eye care professional. 14. Do not use contact lenses used by other people or friend. Only the eye care professional can determine the best product for your eyes. 15. Keep the lenses in room temperature, fresh air and protected from direct sun light while they are in their original packaging. Specific Instructions for Use and Warnings: Instruction for Use in case of Water Activities: Do not expose your contact lenses to water while you are wearing them. WATER CAN HARBOUR MICROORGANISMS THAT CAN LEAD TO SEVERE INFECTION, VISION LOSS OR BLINDNESS. If your lenses have been submerged in water when participating in water sports or swimming in pools, hot tubs, lakes, or oceans, you should discard them and replace them with a new pair. Ask your Eye Care Professional for recommendations about wearing your lenses during any activity involving water. 5. **ADVERSE REACTIONS** Possible Problems Be aware that problems can occur while wearing contact lenses and may or may not be associated with the following symptoms: burning, stinging and/or itchy eyes, reduced lens comfort, feeling of something in your eye (foreign body, scratch and/or area) swelling or inflammation in or around the eyes, eye redness, eyelid problems, watery eyes, unusual eye secretions, poor vision, blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), dry eyes. When any of the above symptoms occur, a serious eye condition may be present. You should immediately be seen by your eye care professional, so that the problem can be identified and treated, if necessary, in order to avoid serious eye damage. 6. **LENS INSERTION/REMOVAL** For your eye health, it is important to carefully follow the handling, insertion, removal, and wearing instructions that we inform below, as well as those prescribed by your eye care professional. 1. Wash your hands thoroughly with a mild soap, rinse completely and dry with a lint-free towel before touching your lenses. Do not use soap with moisturizer. 2. Place one lens at a time in your palm and apply a few drops of multipurpose solution recommended by your eye care professional on the lens surface. 3. Establish a sequence: first clean the lens of the right eye and then the lens for the left eye. 4. With the forefinger of the other hand rub the lens on both sides with gentle movements forwards and backwards, left and right. 5. After cleaning, thoroughly rinse the lenses with multipurpose solution until they are free of residues. 6. Let the lenses rest in the closed case, soaked in multipurpose solution during the period when the lenses are not in use. 7. Clean the lenses every time you use them. 8. Always dispose the multipurpose solution that was inside the lens case when you remove the lenses for use. 9. Do not use heat to disinfect your lenses. 10. The lenses must be cleaned and disinfected every time you remove them from the lens case. One procedure does not substitute the other. Cleaning is necessary to remove mucus and film from the lens surface. Disinfection is necessary to eliminate microorganisms. 7. **LENS HANDLING** 1. Always wash and completely dry your hands before handling the lenses. 2. Gently remove the lenses from the case using the tip of your forefinger and be sure it is clean wet and perfect without any defect. 3. Check if the lens is not upside down; the normal shape is concave with the edges up. If the edges are slightly turned out the lens is inverted. In this case place it in the palm of your hand and gently turn it into the right side. 4. Place the lens on the tip of your forefinger and use the middle finger to pull down your lower eyelid. Using the tip of the other eye, apply the lens. 5. Place the lens on the lens on the eye, gently close the eyelids and blink. Repeat the procedure with the other lens. 6. To remove your lenses, first check in the mirror if they are in place. 7. Look up, slide the contact lens towards the inside corner of the eye using your forefinger and grab it gently between your thumb and forefinger, or follow the method recommended by your eye care professional.

**8. EMERGENCIES** If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into your eyes, SHUT YOUR EYES WITH WATER. RINSE WITH WATER AND IMMEDIATELY CONTACT YOUR EYE CARE PROFESSIONAL OR VISIT A HOSPITAL. EMERGENCY ROOM RIGHT AWAY. Sterile Product: sterilized by heat.

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