HIDROCOR / TORIC HIDROCOR / NATURAL COLORS / TORIC NATURAL COLORS**DAILY USE AND ANNUAL DISPOSAL**

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Rua Domingos Fasolari, 110, CEP (Postal Code) 02513-010, São Paulo-SP, Brazil

Technical responsibility: Roberta Fernandes Dias Jorge,

CRF-SP (São Paulo Regional Pharmacy Association) License No.: 36832

Customer service (international): export@solotica.com.br

Imported by: see product packaging

Expiry date (year/month) and batch: see packaging side

Product sterilized using steam heat. Sterility status indicated on the product label: brown or gray, sterile; pink or purple, not sterile

Contents: contact lenses

Composition: see table below

Quantity: each glass vial contains 1 lens

PRESCRIPTION LENSES, SOLD WITH MEDICAL PRESCRIPTION.

CONSULT YOUR EYE CARE PROFESSIONAL REGULARLY.

STORE ORIGINAL PACKAGING AT ROOM TEMPERATURE.

DISPOSE OF AFTER RECOMMENDED USE.

1. ABOUT THIS USER GUIDE:

The information and instructions contained in this user guide apply to soft contact lenses manufactured by Solótica.

For the health of your eyes, it is important that you use your contact lenses in accordance with the instructions contained in this user guide and any extra care recommended by your eye care professional. They should be fully informed of your medical history so they can develop a complete care program based on your specific needs. Please read this guide carefully and keep it for future reference.

2. ABOUT YOUR CONTACT LENSES AND USE:

Solótica contact lenses are made from biocompatible materials that have the ability to absorb water to keep them soft and flexible. Furthermore, the product uses non-toxic dye to enhance or alter your eye color. These lenses are suitable for daily use (**maximum 8 hours/day**) and **should be disposed annually. These lenses should be replaced and disposed after their useful life (12 months of use).**

***WARNING: Be attentive** to the disposal date of your contact lenses. They should be discarded after the period recommended by the manufacturer. Using lenses after this date can result in **serious eye damage**, including infections and corneal damage, which can progress rapidly and lead to a **loss of vision. RESPECT THE USE PERIOD OF TIME INDICATED BY THE MANUFACTURER.**

For daily reuse, your lenses must be properly sanitized and disinfected before and after use. At the end of the period of daily use of the lenses, store them in an appropriate and properly sanitized lenscase. All this information and/or guidance is provided by your eye care professional and can also be found in this user guide (item 6.3).

3. WHO SHOULD USE CONTACT LENSES (see table below):

Colored prescription lenses: people who have ametropias and who wish to use lenses for corrective.

Family/Model	Product Code			
Natural Colors	0390/0393			
Toric Natural Colors	0400/0413/0414			
Hidrocor	0388/ 0391			
Toric Hidrocor	0399/0411/0412			
Commercial packaging	Colors			
Correction of: Myopia: nearsightedness Hyperopia: farsightedness Astigmatism: difficulty seeing things near and far.	Improved vision	Composition	Use Regime	
		(62% hydroxyethylmethacrylate + 38% water in 0.9% isotonic solution + non-toxic dye)	Daily use, annual disposal	Function
				Benefits
				Results
		Correction of vision problem		

Contact lenses are recommended for daily wear or flexible wear. **Use regime: maximum 8 hours/day. Lenses must be removed before going to sleep.** Replace annually or as recommended by your eye care professional. Always respect the period of use indicated by the manufacturer. After replacement, you should dispose the contact lenses along with organic household waste.

***WARNING: NEVER SLEEP WITH CONTACT LENSES IN.**

***WARNING: Follow** the contact lens daily use regime. Follow the use regime recommended by your eye care professional and the manufacturer. Prolonged daily use of the lenses can result in **serious eye damage**, including infections or corneal damage, which can progress rapidly and lead to a **loss of vision**. **RESPECT THE PERIOD OF INDICATED BY THE MANUFACTURER.**

Contact lenses must be properly cleaned and disinfected, before and after use, using the BIGUANIDA-BASED MULTIPURPOSE CLEANING SOLUTION SPECIFIC TO SOFT CONTACT LENSES recommended by your eye care professional and in accordance with this user guide. The lens should be cleaned and disinfected at least twice a day during the useful life of the product.

4. WHO SHOULD NOT USE CONTACT LENSES:

There are certain health conditions where using contact lenses is not recommended. You should alert your eye care professional to any of the following conditions: *inflammation or infection in or around the eyes or eyelids; *any eye disease, injury or abnormality affecting the cornea, conjunctiva or eyelid; *any previously diagnosed condition that makes wearing contact lenses uncomfortable; *use of any medication that is contraindicated or that interferes with the use of lenses, including eye medication; *severe dry eye condition; *reduced corneal sensitivity; *any systemic disease that may be exacerbated or interfere with the safe use, handling and/or care of the lens; *any allergic reaction on the surface of the eye or surrounding tissues that may be induced or aggravated by the components of the product or Biguanida-based multipurpose cleaning solution specific to soft contact lens. Only a specialist can determine if using contact lenses is right for you.

5. THE FOLLOWING PROBLEMS MAY OCCUR WITH THE USE OF CONTACT LENSES:

Common: Stinging, burning sensation, itching or tearing; *discomfort and irritation when wearing lenses (2%); * foreign body sensation in the eyes, *red eyes; *dry eyes.

Uncommon: *Poor and blurred vision (0.1%); *rainbow vision or halos around objects; *sensitivity to light.

Rare: *Swelling or inflammation in the eyes or in the surrounding area of eyes; *pain, * infectious keratitis, non-infectious keratitis, * decreased corneal sensitivity.

What you should do if a PROBLEM occurs. If any of the above symptoms or problems occur, IMMEDIATELY REMOVE THE LENS and consult your eye care professional. Any serious symptoms and/or adverse events that have occurred in relation to the use of the contact lens must be reported to the manufacturer via the SAC email and to the competent authority of the Member State in which the wearer and/or patient is established.

6. HANDLING, CLEANING AND PUTTING ON LENSES

For good eye health, IT IS ESSENTIAL TO FOLLOW THE GUIDELINES OF YOUR EYE CARE PROFESSIONAL AND FOLLOW CLOSELY THE INSTRUCTIONS FOR HANDLING THE LENS PACKAGING AND FOR LENS CLEANING, PUTTING ON, REMOVAL AND USE CONTAINED IN THIS USER GUIDE. If you are unable to follow the recommended care measures, **YOU SHOULD NOT USE CONTACT LENSES.**

6.1. HANDLING THE LENS PACKAGING

Step 1: Check the expiry date of the product. If it has passed the expiry date, **DO NOT USE THE PRODUCT**. Contact your eye care professional and the place where you purchased the lens.

Step 2: Check the sterility of the product. **HOW?** Check the sterility status indicated on the product label: brown or gray, sterile; pink or purple, not sterile. **The product should not be used if it is pink or purple.** In this case, contact your eye care professional and the place where you purchased the lens.

Step 3: After the above checks, the packaging should be kept at room temperature and out of direct sunlight. **DO NOT USE THE PRODUCT IF THE ORIGINAL PACKAGING HAS BEEN VIOLATED OR DAMAGED.** Contact your eye care professional and the place where you purchased the lens.

6.2. LENS HANDLING

Step 1: Starting: It is essential that you adopt good hygiene practices in the care and handling of your lenses. Adopt a proper hand hygiene routine. You should wash and DRY your hands before handling lenses to remove any residues from cosmetics, soaps, lotions or creams.

***WARNING: Avoid** using oily cosmetics, such as moisturizing soaps, lotions or creams. These substances can come into contact with the lens and cause discomfort and/or irritation. Before opening the original packaging or touching the lenses, wash your hands with mild soap, rinse and dry with a lint-free towel. **Do not touch your contact lenses with your fingers or hands if they are not completely clean and dry. Do not apply cosmetics, lotions, soaps, creams or hair sprays before handling your lenses. Contact lenses must be put on and removed before applying and removing makeup. Water-based cosmetics are recommended as they cause less damage to lenses than other products.**

***WARNING: Pay attention to vapors, such as makeup fixing sprays, as these can be harmful and irritating. Avoid using them when wearing lenses, or keep your eyes closed and protected when applying this type of spray.**

Step 2: Opening the packaging: Each glass vial stores a single lens. You can keep the lens inside the vial until ready to use. To open the vial, follow the instructions. First, gently shake the vial and check if the lens is floating in the solution. If the lens remains stuck to the lid or bottom of the vial, keep shaking until it is free. Then carefully lift and break the seal flap.

***WARNING: BE CAREFUL WHEN REMOVING THE SEAL TO AVOID CUTTING YOUR FINGER AND/OR HANDS. WHILE REMOVING THE SEAL, THERE MAY BE PARTS THAT CAN CAUSE ACCIDENTS.**

Remove the seal and the silicone cap. Pour the contents of the vial onto one of your hands. Pick up the lens with your fingertips and place it in the lenscase. If the lens remains in the bottle, add a sufficient amount of Biguanida-based multipurpose cleaning solution specific to soft contact lens, cap and gently shake the vial again to float the lens and repeat the operation.

Step 3: Handling the lens: When using your lenses for the first time, make sure that you are able to put on and remove the lenses by yourself or have a person nearby who can do this for you. Always handle your lenses with your fingertips, as fingernails or other instruments can scratch the lens surface and cause discomfort. Use your fingertips to gently slide the lens out of the case until the lens is on your finger.

***WARNING: NEVER USE tweezers, fingernails, or any other instrument to remove the lens from the eye or lenscase.** This can damage or even tear the contact lens, causing discomfort or eye damage.

Step 4: Inspect the lens: After removing the lens from the packaging or lenscase, examine it to make sure the lens is damp, clean and free of nicks or tears.

***WARNING: Be attentive! IF THERE IS ANY SIGN OF DAMAGE, DO NOT USE THE LENS.** Damaged lenses may cause discomfort and pain during use. Store the lens in the case in a multi-purpose solution and contact your eye care professional.

Step 5: Handling habits: Develop the habit of always handling the lens of a given eye first to avoid using the wrong lens, resulting in blurred vision.

6.3. CLEANING YOUR CONTACT LENSES

Step 1: Check that the MULTI-PURPOSE CLEANING SOLUTION you purchased is SPECIFIED FOR SOFT CONTACT LENS and BIGUANIDA-BASED. Also verify the expiry date. If expired, dispose of the solution. DO NOT USE EXPIRED SOLUTION on your contact lenses.

***WARNING: Pay attention** when purchasing MULTI-PURPOSE SOLUTION as it can cause irritation and eye allergies.

Step 2: Place one lens at a time in the palm of your hand and apply a few drops of the multi-purpose solution recommended by your eye care professional to the surface of the lens. With the index finger of the other hand, rub the lens on both sides with smooth movements in different directions. After cleaning, rinse the lens thoroughly with the Biguanida-based multipurpose cleaning solution specific to soft contact lens and ensure it is free of residue. Always handle lenses with care and avoid dropping them. If your lens falls, clean it again.

***WARNING: Be attentive! ALWAYS CLEAN LENSES BEFORE AND AFTER USE.** When storing or handling lenses, always use new solutions within the expiry date. Strictly follow the solution instructions for use.

NEVER CLEAN YOUR LENSES WITH RUNNING WATER (THIS INCREASES THE RISK OF INFECTION) OR COMMON SALINE SOLUTION (THIS IS NOT SUITABLE FOR CLEANING OR DISINFECTING).

6.4. PUTTING ON YOUR LENSES

Step 1: Check the lens position: After cleaning and inspecting, verify that the lens is in the correct position. The normal shape of the lens is concave with the edges up. If the edges are facing down slightly, the lens is inverted. **AN INVERTED LENS WILL CAUSE IRRITATION AND BLURRED VISION.** In this case, place it in the palm of your hand and gently turn it over to the correct side.

Step 2: Putting on the lens: After checking the lens position, put the lens on the tip of your index finger and use your middle finger to pull down your lower eyelid. Using the middle finger of the other hand, lift the upper eyelid. Place the lens on the eye, gently release the eyelids and blink. Repeat the operation for the other lens. After the lenses are on your eyes, make sure your vision is normal and the lenses are comfortable. If you are experiencing discomfort or blurred vision, it could be that the lens is on the wrong eye, dirty or inverted. **IT IS IMPORTANT TO INSPECT THE LENS** to make sure it is not damaged. If the lens is intact, repeat the cleaning procedure before putting it back on. If the problems persist after it, **REMOVE THE LENS IMMEDIATELY** and contact your eye care professional.

***WARNING:** If the lens is damaged, **DO NOT** use it again. Dispose it, replacing it with a new one, or see your eye care professional.

***WARNING:** If the lens is not properly centered, gently blink until the lens slides into the correct position. If the lens does not move, close your eye and gently massage the eyelid until it is centered. If the lens is still not correctly positioned, remove and store it and contact your eye care professional.

***WARNING:** If you experience the sensation of the lens stuck to your eye, do not try to remove it, as this could harm your eyes. In this case, drip a few drops of a lubricating solution recommended by a specialist. If it does not improve, **CONTACT YOUR EYE CARE PROFESSIONAL IMMEDIATELY.**

6.5. USE OF LENSES

When wearing contact lenses, remember the following precautions:

- * Avoid wearing lenses in environments subject to harmful chemical vapors and/or irritants.

- * It is not recommended to wear contact lenses when bathing or playing water sports due to potential contamination of the lens with microorganisms present in the water. In this case, dispose and replace them with a new pair. This contamination can lead to severe infections, partial loss of vision or even blindness. Ask your eye care professional for recommendations on wearing lenses during water activities.

- * NEVER clean or store your lenses in any liquid other than a multi-purpose solution SPECIFIED FOR SOFT CONTACT LENSES and recommended by a specialist. Other solutions can lead to contamination of your lenses by *Acanthamoeba*, a microorganism associated with the occurrence of infectious keratitis.

- * Do not heat the disinfection solution or lenses.

NEVER SHARE YOUR CONTACT LENSES.

- * If the lens tears or breaks while on your eye, carefully remove the pieces the same way you would remove the entire lens. If the pieces do not come off, do not touch your eye. Rinse with sterile saline solution to try to remove any pieces. If necessary, consult a specialist.

- * The sensation of dry eyes during use can be relieved by increasing the frequency of blinks or by using lubricating or moisturizing drops indicated by your eye care professional. If dryness persists, consult a specialist.

- * It is possible that the edges of the lenses can stick together. Do not try to separate them abruptly to avoid tearing. Add the Biguanida-based multipurpose cleaning solution specific to soft contact lens and slowly move back and forth until they come apart.

6.6. LENS REMOVAL AND LENS CASE CARE

Step 1: Lenscase handling: Always have a new multi-purpose solution in the lenscase. Whenever you store or handle your lenses, use a new solution within the expiry date. After removing the lens, clean the lenscase with the remaining solution by rubbing it. Dispose of the solution and let the lenscase dry. Follow the instructions for using the solution.

Step 2: Hand cleaning: Clean your hands according to step 1 of item 6.2 Lens Handling.

Step 3: Removing lenses: Look up, gently slide the lens towards the outside of your eye with your index finger and grasp it between your thumb and forefinger, or follow the method recommended by your eye care professional.

***WARNING:** Be careful to use only your fingertips as described in the WARNING of step 3 of item 6.2. Lens Handling.

Step 4: Lens cleaning: Clean the lenses as described in item 6.3. Contact Lens Cleaning.

Step 5: Lens storage: After cleaning, gently take the lens with your fingertips and place it in the lenscase, submerging it in the solution and ensuring it is away from the edge of the lenscase to avoid dehydration and unexpected tears. Leave the lenses in the closed lenscase immersed in a Biguanida-based multipurpose cleaning solution specific to soft contact lens when not in use. Always use the appropriate lenscase. If you do not use your lenses daily, do the full cleaning procedure once a week.

***WARNING:** Always put enough solution IN THE LENS CASE to cover the lens.

***WARNING:** If the lenscase is stored in a place where it can flip over unintentionally (such as a toiletry bag), the multi-purpose solution may leak out, allowing the lens to dry out and stick to the lenscase. In this case, rehydrate the lenses by

leaving them immersed in a Biguanida-based multipurpose cleaning solution specific to soft contact lens for approximately 1 hour. If you notice any change in appearance, or if it feels uncomfortable, dispose of the lens immediately.

***WARNING: It is recommended to have spare lenses, lenscase, cleaning solution and glasses.**

6.7. TAKING CARE OF THE LENS CASE:

Cleaning the lenscase: Cleaning the lenscase is just as important as cleaning your lenses. One procedure does not substitute the other. Wash the lenscase with a Biguanida-based multipurpose cleaning solution specific to soft contact lens and let it dry at room temperature, **IMMEDIATELY AFTER PUTTING ON THE LENSES ON YOUR EYES.**

***WARNING:** NEVER clean or store the lenscase in any liquid other than a Biguanida-based multipurpose cleaning solution SPECIFIC TO SOFT CONTACT LENS and recommended by a specialist. Other solutions can lead to contamination of your lenscase by *Acanthamoeba*, a microorganism associated with the occurrence of infectious keratitis.

***WARNING:** Do not leave the solution in the lenscase without the lens to avoid the growth of microorganisms. Always dispose of the multipurpose solution.

***WARNING:** It is very important to replace the lenscase at regular intervals (approximately 3 months).

7. WARNING: Misuse of contact lenses and lens care products can result in **serious eye damage**. Serious problems such as infections or corneal injuries may occur and result in **vision loss**. Studies show that contact lens users who smoke and/or sleep with their lenses are more likely to develop serious eye problems⁽¹⁾. **Diabetics may have reduced corneal sensitivity** and are more prone to corneal damage.

***WARNING:** Soft contact lenses are not indicated for therapeutic use as a bandage for refractive surgery.

8. GENERAL PRECAUTIONS:

***Note** that certain medications, such as anti-allergic drugs, decongestants, diuretics, muscle relaxants, tranquilizers and motion sickness medicines, can cause dry eyes, greater perception of contact lenses (the lens is more noticeable on the eye) or blurred vision⁽¹⁾. Always inform your eye care professional about the use of any medications.

***WARNING:** Oral contraceptive users and pregnant women⁽²⁾ may develop changes in vision quality and comfort while wearing contact lenses.

9. STORAGE CARE: Keep the lenses between 15 °C to 25 °C and out of direct sunlight while in the original packaging.

10. EMERGENCY: If chemicals of any kind accidentally come in contact with your eyes, **IMMEDIATELY RINSE YOUR EYES WITH RUNNING WATER AND REMOVE THE LENS. CONTACT YOUR EYE CARE PROFESSIONAL OR GO TO THE HOSPITAL.**

11. TABLE OF SYMBOLS.

Symbol	Description
	Read the instructions
	Made in/by
	Manufacturing date
	Expiry date
	Batch number
	Product sterilized by steam heat and unique external Barrier System
DIA	Diameter
BC	Base curve
PWR	Lens power
	Single patient – Multiple uses
	European certification and authorized representative details.
	Product made in Brazil
	Do not re-sterilize
	Temperature limit
	Catalog number
	Serial number/Manufacturer's serial number
	Associated precautions
	Authorized Representative in the European Community
	Do not use if packaging is damaged
	Avoid exposure to sunlight
	Keep dry
	Lens identification code

	Sold by prescription
	Importer

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